




Steps for Use:

1. Press Power button



2. Insert documents:
Top Down
Face Down

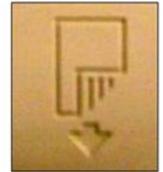

3. Log In to AFHCANcart

4. Select the Patient button



5. Search for the patient

6. After searching:

Choose either  or  to create a case for the patient

7. Verify patient name 

AFHCAN CART
 Patient: Castle, Chris (64 yrs, M)
 Case #: Med North-E-8
 User: Jenny Perker Castle, Chris (64 yrs, M)
 Site: Med North Cart One
 AFHCANcart 8.0
 Med North Working On-Line
 0 Cases in Transit
 About

Buttons Explained:




Press Black & White, Grayscale or Color buttons to scan documents. Grayscale & Color are higher quality.

Rotate scans before saving

Save scans to the case




Single Attachment- Combines multiple pages into one attachment

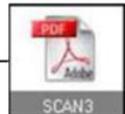


Separate Attachment- Saves each page as it's own pdf.



To Save:

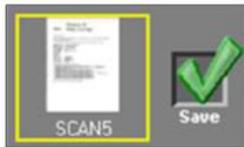

Label

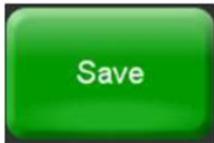

Click on attachments to check scan quality. Rescan in Grayscale or Color if poor or unreadable.

